

Gold Plated & Titanium Screw Posts



CONSULT
INSTRUCTIONS
FOR USE



Sterilize Before Use



Screw Posts are
Single Use Only

Rx Only in USA

For Dental Use Only

Non-Sterile Product. Autoclave before use at 136° C for 20 minutes.

INSTRUCTIONS FOR USE

Screw Posts are used to give retention for reconstruction of non-vital teeth, where the root canal has been endodontically treated.

The gold plated screw posts should not be shortened or tampered with in any way that might take away part of the protective gold layer as this can cause the post to corrode. Titanium or stainless steel posts are not affected by this.

It is of outmost importance that the root-canal has been prepared with a reamer of appropriate size and shape corresponding to the post to be used.

When using the reamers, start with the smallest reamer and proceed with larger and larger reamer until the desired size is reached. Reamers are numbered 1 through 6 and correspond directly to the post sizes. Reamers should be used at moderate r.p.m. speed with a careful up and down movement only. If these precautions are not taken, the reamer can get stock and/or break off in the root canal.

Never use force when inserting the screw post as this can cause root fracture.

Use appropriate cement to increase retention of screw post. Screw posts should be used by trained professionals only.

Screw posts and reamers can be sterilized by all methods. Autoclave before use at 136°C for 20 minutes. Reamers are re-usable when cleaned and re-sterilized.

Manufactured for JS Dental by
Harald Nordin SA